

Shipping a PGT Specimen to Millennium Health[™]

Please follow these instructions closely to ensure specimens are packaged properly and include all required documents.



Important:

DO NOT ship specimens to Millennium in UN3373 CAT B packaging. Use only materials provided by Millennium Health.



TIGHT!

Firmly tighten the blue cap on the specimen collection device. Place bar code sticker from test requisition vertically on the device.



Put one specimen collection device into the right hand side of the bag marked "Exempt Human Specimen" with absorbent pad inside.




Fold Test Requisition form in half twice, leave top right corner with specimen ID facing out. Place Test Requisition form and patient information sheet into the left hand side of the specimen bag marked "Test Requisition" and seal.



Place up to eight specimen bags into the green colored inner bag. Please do not place any other specimen types (urine) with saliva or buccal specimens.



Place up to three inner bags into the overnight shipping bag. Seal the bag with adhesive strip.

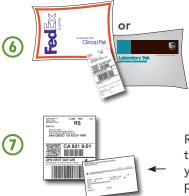

Apply label to UPS or FedEx shipping bag.

Retain label tag (with tracking information) for your records and release package to UPS/FedEx.

TO SCHEDULE A PICKUP

Via MLIS (UPS Clients only)

- Enable "Services" tab via Sales Support (client email address required)
- Login to MLIS > Services > Schedule Specimen Pickup
- Choose date/time, enter contact name/ phone, and submit

Via Phone/Email

- Call 858.217.0322 or email distribution@millenniumhealth.com
- Provide complete practice information, contact information and requested pickup time

Via Daily Pickup/Automation (High Volume Clients Only)

- Shipping 4–5 days/week minimum
- 24–48 hour lead time prior to set up
- Call/email to configure

Customer Service 877.451.3534 millenniumhealth.com

The Millennium PGT logo is a service mark or registered service mark of Millennium Health, LLC or its subsidiaries in the United States and other countries. All other trademarks used herein are the property of their respective owners. MLI-MKT38005