

INDICATIONS FOR USE - When to use the IceMan CLASSIC³

The intended use of the IceMan[®] is for the temporary reduction of swelling and pain after surgery or injury. This is a non-sterile prescription device for single patient use to provide localized external application of cold therapy.

CONTRAINDICATIONS - When not to use the IceMan CLASSIC³

DO NOT use this device on patients with Raynaud's phenomenon or other vasospastic conditions; Buerger's disease; cold allergy or hypersensitivity; cryoglobulinemia; paroxysmal cold hemoglobinuria or other cold agglutinin disorders; pheochromocytoma; sickle cell anemia or history of cold injury.

WARNING THIS PRODUCT CAN BE COLD ENOUGH TO CAUSE SERIOUS INJURY

WARNING! Read and understand all warnings and Instructions for Use before using this device. Additional warnings appear in the Instructions for Use.

WARNING! DO NOT use this device without a prescription from a physician. Rx only. Your prescription must state how long and how often the device should be used and the length of breaks between uses. **DO NOT** use this device if a prescription has not been provided to you or if you do not understand the prescription.

WARNING! This device can be cold enough to cause serious injury, including tissues necrosis. You must be able to check your skin condition under the cold pad. **DO NOT** use if you cannot check your skin condition frequently (at least every hour). Check for increased pain, burning, numbness, tingling, increased redness, discoloration, itching, increased swelling, blisters, irritation or other changes in skin condition under the cold pad or around the treatment area. If you experience any of these conditions, immediately discontinue use of this device and contact your physician.

PRECAUTIONS - When to exercise special care when prescribing the IceMan CLASSIC³

Exercise special care prescribing this device for the following patients: those with arthritic conditions; peripheral vascular disease; children under 12; those with decreased skin sensitivity, poor circulation or compromised local circulation; hypercoagulation disorders; diabetes or neuropathies.

DonJoy IceMan CLASSIC Medical Equipment
DO NOT use any other wall plug/power supply with this device other than the one provided.



IP21



5919 Sea Otter Place, Suite 200 | Carlsbad, CA 92010 | U.S.A.



DJO, LLC | A DJO Company

DJOglobal.com



only



IceMan and DonJoy are trademarks of DJO LLC
U.S. Patent
5,865,841 / 5,980,561
Made in Mexico
LBL-11-0500-DV2 Rev C
Copyright © by DJO, LLC

Pantone Cool Gray 11C

Pantone 422C